



R_x Only
Genadur[®]
Hydrosoluble Nail Lacquer



R_x Only
For Topical Use Only

DESCRIPTION

Genadur[®] is a hydrosoluble nail lacquer.

INDICATIONS AND USES

Genadur[®] is indicated to protect intact or damaged nails from the effects of moisture, friction (rubbing) or shear (tearing), relieving symptoms and signs of nail dystrophy (i.e. nail splitting and fragility).

CONTRAINDICATIONS

Genadur[®] is contraindicated in persons with known hypersensitivity to any components of the formulation.

WARNINGS

Do not use on open wounds. Do not ingest.

FOR EXTERNAL USE ONLY. NOT FOR OPHTHALMIC USE.

KEEP THIS AND ALL MEDICATIONS OUT OF THE REACH OF CHILDREN.

DIRECTIONS

Apply **Genadur**[®] once daily to clean and dry nails. Using the supplied brush, apply a thin layer of the product on the affected nail(s). Close bottle immediately after use. The film is hydrosoluble, therefore **Genadur**[®] should be applied after nail washing and drying, preferably before bedtime. **Genadur**[®] may also be used under a cosmetic colored nail polish, thus protecting the nails from the damaging action of solvents used for polish removal.

INGREDIENTS

Diethylene glycol monoethylether, *equisetum arvense* extract, ethanol, hydroxypropyl chitosan, methyl sulfonyl methane, water.

HOW SUPPLIED

Genadur[®] Hydrosoluble Nail Lacquer is available in a 12 mL bottle with applicator brush, 43538-510-12.

Store at 15°C to 30°C (59°F to 86°F). Protect from heat.

Mfd. for: Medimetriks Pharmaceuticals, Inc., 383 Route 46 West, Fairfield, NJ 07004-2402 USA

Mfd. by: Polichem S.A., Via Senago 42D, 6912, Lugano-Pazzallo, Switzerland

Made in Italy

RX - Prescription Medical Device: Federal Law restricts this device to sale by or on the order of a physician.



US Patent: 8,318,214 B2

Rev: 2/16



383 Route 46 West • Fairfield, NJ 07004-2402 USA • www.medimetriks.com

IP021-R3